



GIRAFFE
BOTTLE

HYDRATION WITH A PURPOSE™

**ASSISTIVE
DRINKING SYSTEM**

CARE & USE GUIDE

...FEATURING THE JOURNEY NECK



SCAN ME



**TRANSLATIONS
AVAILABLE**

ES FR IT DE

WWW.GIRAFFEBOTTLE.COM/CARE-AND-USE-GUIDES

Components + Care

Tritan™ + Stainless Steel Bottle Kits




DO NOT PLACE
BOTTLES IN
FREEZER/MICROWAVE

Recommended Max Temperature: 185°F / 85°C




SAFE FOR
DISHWASHER

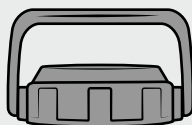

NOT SAFE FOR
DISHWASHER



HAND-WASH
ONLY

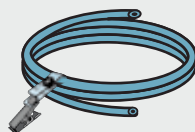


Drinking Tube + Bite Valve

The bite valve must be removed from the drinking tube during cleaning and should **NOT** be pierced with a brush.



USE INCLUDED
CLEANING BRUSH

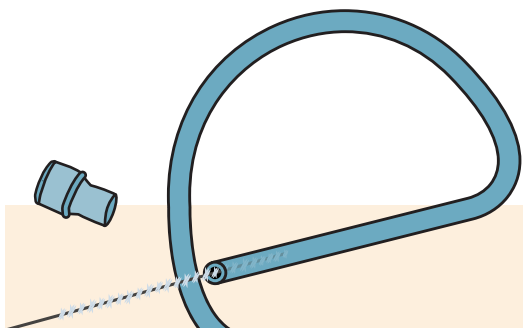


If you have a taste sensitivity to the supplied drinking tube (after washing), inquire for an alternate tube

Journey Neck Assembly

A removable clip can attach to clothing and fabric when not in use, without risk of leaks.





TUBE CAN BE CUT TO SIZE IF NEEDED

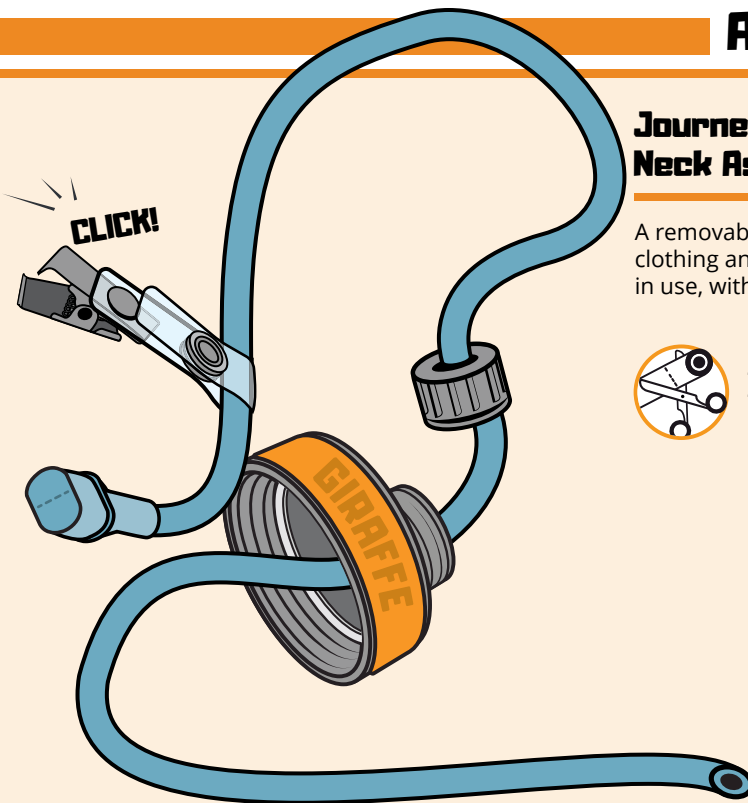

If you are concerned the bite valve will present a hazard as a small object, remove from the drinking tube during use

Multiple Mounts

Pair Journey Systems with Bottle Holders, or a Carry Bag for chairs, shoulders, etc.

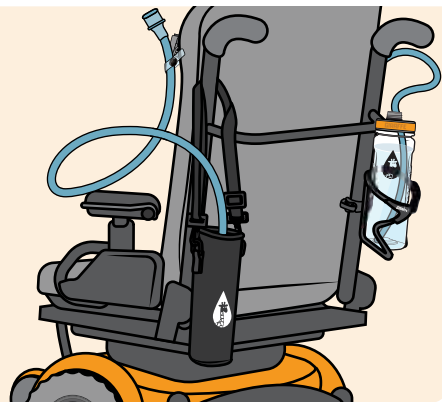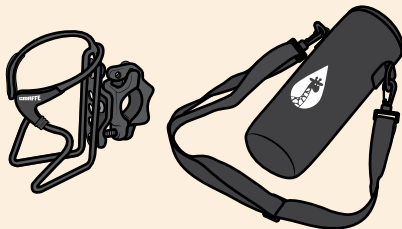

REPLACEMENT SCHEDULE

DRINKING TUBE MONTHLY

CLEANING BRUSH . . EVERY 3 MONTHS

BITE VALVE MONTHLY


Replace integral parts of the Giraffe Bottle Journey System regularly to maintain optimal bottle hygiene and function.



Recommended Use

Which Beverages Can You Drink?

Giraffe Bottle is recommended for water and other **non-viscous** liquids. Remove the bite valve when enjoying other beverages.

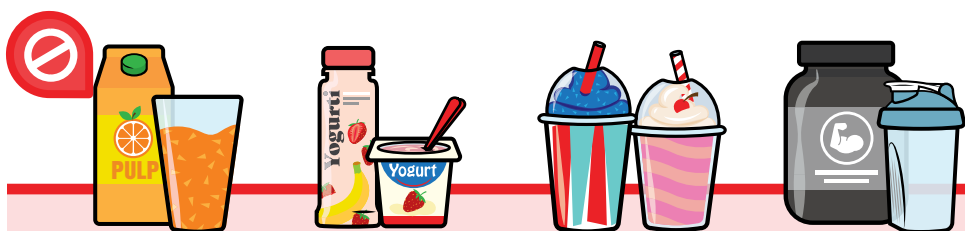

! The bite valve may become clogged if used with thicker beverages

Avoid Leaks

The Journey System is free-standing, and perfect for a variety of active applications. These include, but are not limited to: post-op care, postpartum recovery, kayaking, hiking, biking, and more.

To avoid leaks during use, **do not** place the bottle completely on its side, or flip upside down.



! This system is recommended for individuals with good control over mouth functions

THE JOURNEY SYSTEM

NECK LENGTHS AVAILABLE

36 Inch (91 cm)

*Can be cut to length if needed

DRINKING TUBE TYPE

Large Flexible Tube

1/4" (6.4mm) Inner Diameter

FEATURED LIQUID CONTROL

Bite Valve

*Removable from Tube

BOTTLE HOLDER OPTIONS

Aluminum Bottle Holder /
Bottle Carry Bag

THIS SYSTEM IS BEST FOR:

Individuals with limited upper
body mobility, combined with
full lung and mouth function.

LEVELS OF

HYDRATION

Giraffe Bottle offers a few unique options for Hands-Free Hydration. Each one is tailored for different needs and purposes, with experiences of individuals in mind.

Our systems are cross compatible with every bottle; explore them all and see which fits your needs best!



FAQ'S

Questions about your
current system?

WWW.GIRAFFEBOTTLE.COM/PRODUCT-FAQ

LEARNING CENTER

Check out the Learning Center on **GiraffeBottle.com** to explore Hands-Free Hydration, and Frequently Asked Questions about our products. Feel free to contact Giraffe Bottle with any questions, comments, or feedback.




Giraffe Bottle Accessories



Replacement Giraffe Bottles

Tritan Clear: #504111
Smoke Black: #504211
Burnt Orange: #504212
Ltd. Ed. SGN Pattern: #504221

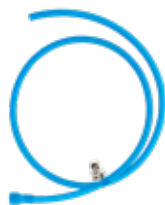

Drinking Tube + Bite Valve

Replacement tube/valve combo for Journey systems.

#505181



Bottle Carry Bag w/ Strap

Carry bottles on shoulders, looped over chairs, and more.

#504062



Clamp Mount Bottle Holder

Mounts to wheelchairs and other round tubing.

#500311



Cushion Companion Bottle Holder

Slide under mattresses, seat cushions, and other surfaces.

#500332



Weighted Desktop Base

Keep bottles secure and upright on flat surfaces with this sturdy bottle holder.

#500323



Rail Mount Bottle Holder


Attaches to powerchairs with accessory rails (includes Quantum™, Permobil™).

#500314

More Available Online

For alternate drinking tubes (due to sensitivity in taste), accessories, and other replacement parts, explore online or contact Giraffe Bottle.

WWW.GIRAFFEBOTTLE.COM

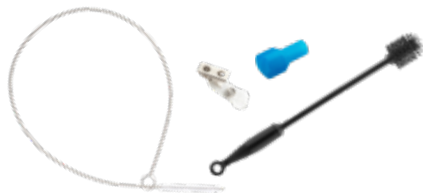

SHOWN: JOURNEY TUBE BRUSH, SILICONE BOTTLE BRUSH, BITE VALVE, TUBE CLIP



GIRAFFE
BOTTLE

a division of:
Oak Hill
BRANDS CORP.

SALES@GIRAFFEBOTTLE.COM

1-630-922-5004

Product designed and assembled in the USA. Manufactured with U.S. and global components.

Information and specifications listed are subject to change at any time without notice.

Permobil, Quantum, and all other brands that may have been mentioned, either directly or indirectly, are trademarks of someone else and are not the property of Oak Hill Brands. Giraffe Bottle is a registered trademark of Oak Hill Brands Corp.

v2312