



GIRAFFE
BOTTLE

HYDRATION WITH A PURPOSE™

**HANDS FREE
DRINKING SYSTEM**

CARE & USE GUIDE

...FEATURING THE TOWER XL NECK



SCAN ME



**TRANSLATIONS
AVAILABLE**

ES FR IT DE

WWW.GIRAFFEBOTTLE.COM/CARE-AND-USE-GUIDES

Components + Care



Tritan™ + Stainless Steel Bottle Kits





**DO NOT PLACE
BOTTLES IN
FREEZER/MICROWAVE**

Recommended Max Temperature: 185°F / 85°C




**SAFE FOR
DISHWASHER**

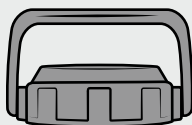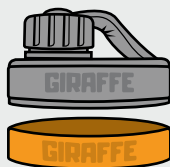

**NOT SAFE FOR
DISHWASHER**



**HAND-WASH
ONLY**

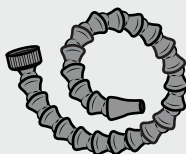


**Avoid submerging modular neck during
cleaning— wipe down outer surface**

Drinking Tube + Bite Valve

The bite valve must be removed from the drinking tube during cleaning and should **NOT** be pierced with a brush.




**USE INCLUDED
CLEANING BRUSH**

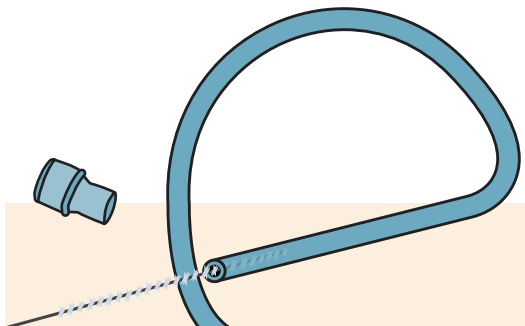


**If you have a taste sensitivity to the supplied drinking
tube (after washing), inquire for an alternate tube**

Tower XL Neck Assembly

The assembly is designed to **POP** apart to adjust neck length as needed.

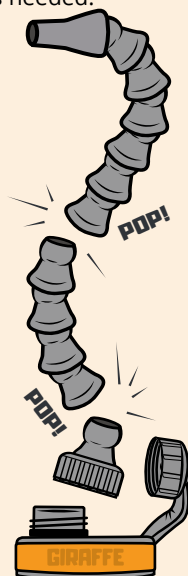

If you are concerned the bite valve will present a hazard as a small object, remove from the drinking tube during use

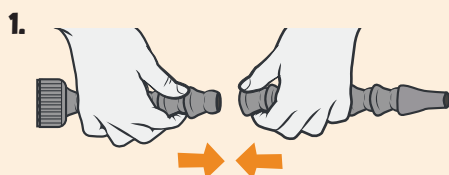

REPLACEMENT SCHEDULE

DRINKING TUBE MONTHLY

CHECK VALVE MONTHLY

CLEANING BRUSH . . . EVERY 3 MONTHS

NECK ASSEMBLY . . . EVERY 8 MONTHS


Replace integral parts of the Giraffe Bottle Tower XL System regularly to maintain optimal bottle hygiene and function.



Recommended Use

Which Beverages Can You Drink?

Giraffe Bottle is recommended for water and other **non-viscous** liquids. Remove the bite valve when enjoying other beverages.

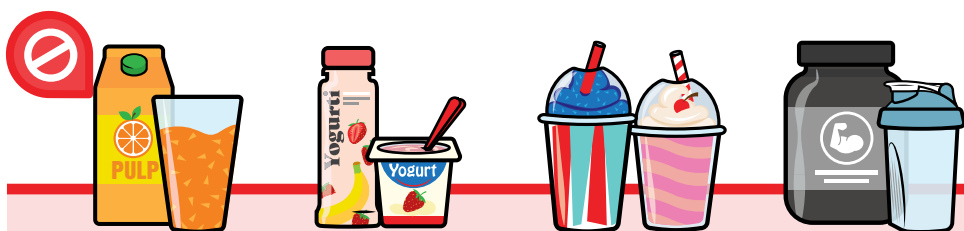


! The bite valve may become clogged if used with thicker beverages

Control the Flow

Pressing the bite valve between two surfaces (biting motion) allows it to open. Combined with suction, you can control the flow of your drink and know it won't leak outside of use.

With the bite valve in place, your drink won't retreat back into the bottle after taking a sip. If the bite valve is not necessary for your application, it can easily be removed.

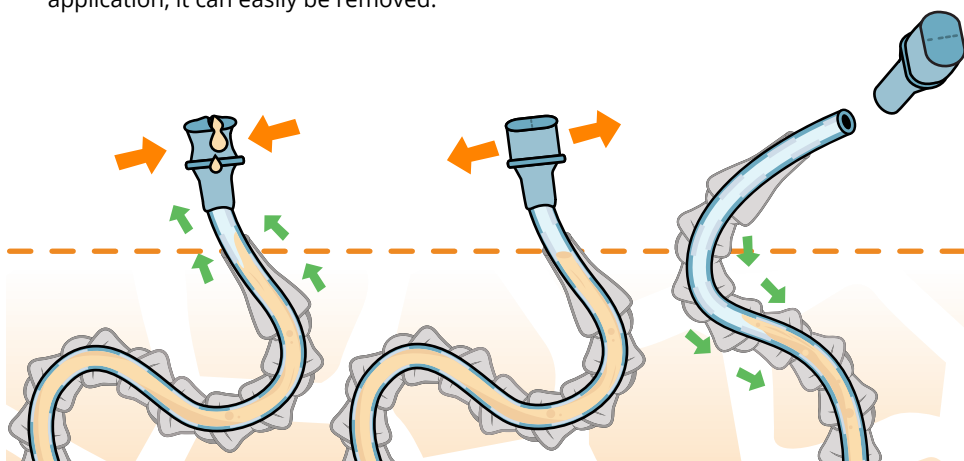

! This system is recommended for individuals with good control over mouth functions

THE TOWER XL SYSTEM



NECK LENGTHS AVAILABLE

15", 25", 30", 36"
(38 cm, 64 cm, 76 cm, 91 cm)

DRINKING TUBE TYPE

Large, Housed in Modular Plastic
1/4" (6.4mm) Inner Diameter

FEATURED LIQUID CONTROL

Bite Valve
*Removable from Tube

BOTTLE HOLDER OPTIONS

Aluminum Bottle Holder

THIS SYSTEM IS BEST FOR:

Individuals with full lung and
mouth function, but little-to-no
upper body mobility.

LEVELS OF

HYDRATION

Giraffe Bottle offers a few unique options for Hands-Free Hydration. Each one is tailored for different needs and purposes, with experiences of individuals in mind.

Our systems are cross compatible with every bottle; explore them all and see which fits your needs best!



FAQ'S

Questions about your
current system?

WWW.GIRAFFEBOTTLE.COM/PRODUCT-FAQ

LEARNING CENTER

Check out the Learning Center on **GiraffeBottle.com** to explore Hands-Free Hydration, and Frequently Asked Questions about our products. Feel free to contact Giraffe Bottle with any questions, comments, or feedback.



Giraffe Bottle Accessories



Replacement Giraffe Bottles

Tritan Clear: #504111
Smoke Black: #504211
Burnt Orange: #504212
Ltd. Ed. SGN Pattern: #504221



Drinking Tube + Bite Valve

For 15" Neck: #504072
For 25" Neck: #504074
For 30" Neck: #504075
For 36" Neck: #504076



Tower XL Starter Kit

15" Neck: #505162
25" Neck: #505164
30" Neck: #505165
36" Neck: #505166



Clamp Mount Bottle Holder

Mounts to wheelchairs and other round tubing.

#500311



Cushion Companion Bottle Holder

Slide under mattresses, seat cushions, and other surfaces.

#500332



Weighted Desktop Base

Keep bottles secure and upright on flat surfaces with this sturdy bottle holder.

#500323



Rail Mount Bottle Holder

Attaches to powerchairs with accessory rails (includes Quantum™, Permobil™).


#500314



More Available Online

For alternate drinking tubes (due to sensitivity in taste), accessories, and other replacement parts, explore online or contact Giraffe Bottle.

WWW.GIRAFFEBOTTLE.COM

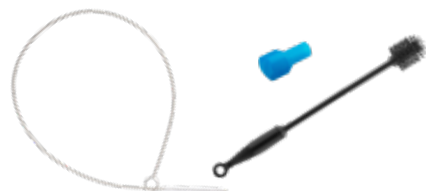

SHOWN: TOWER XL TUBE BRUSH, SILICONE BOTTLE BRUSH, BITE VALVE



GIRAFFE
BOTTLE

a division of:

Oak Hill
BRANDS CORP.

SALES@GIRAFFEBOTTLE.COM

1-630-922-5004

Product designed and assembled in the USA. Manufactured with U.S. and global components.

Information and specifications listed are subject to change at any time without notice.

Permobil, Quantum, and all other brands that may have been mentioned, either directly or indirectly, are trademarks of someone else and are not the property of Oak Hill Brands. Giraffe Bottle is a registered trademark of Oak Hill Brands Corp.

v2312