



GIRAFFE
BOTTLE

HYDRATION WITH A PURPOSE™

**HANDS FREE
DRINKING SYSTEM**

CARE & USE GUIDE



...FEATURING THE TOWER NECK

SCAN ME



**TRANSLATIONS
AVAILABLE**

ES FR IT DE

WWW.GIRAFFEBOTTLE.COM/CARE-AND-USE-GUIDES

Components + Care

Tritan™ + Stainless Steel Bottle Kits





DO NOT PLACE
BOTTLES IN
FREEZER/MICROWAVE

Recommended Max Temperature: 185°F / 85°C





SAFE FOR
DISHWASHER

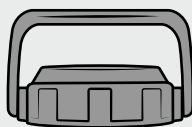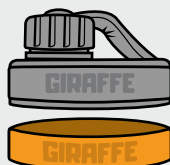

NOT SAFE FOR
DISHWASHER



HAND-WASH
ONLY

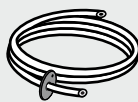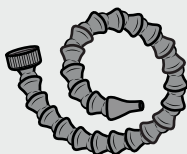


Avoid submerging modular neck during
cleaning— wipe down outer surface

Drinking Tube + Check Valve

The check valve must be removed from the
drinking tube during cleaning and should
NOT be pierced with a brush.




USE INCLUDED
CLEANING BRUSH

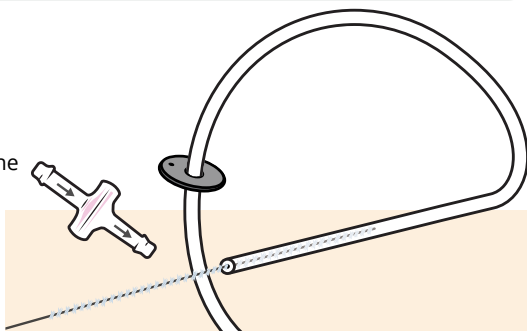

If you have a taste sensitivity to the supplied drinking
tube (after washing), inquire for an alternate tube

Tower Neck Assembly

The assembly is designed to **POP** apart to adjust neck length as needed.

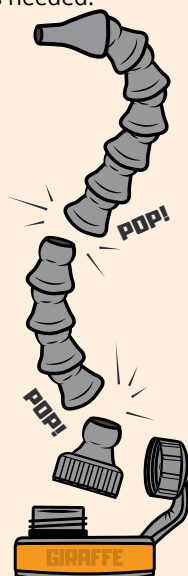

REPLACEMENT SCHEDULE

DRINKING TUBE MONTHLY

CHECK VALVE MONTHLY

CLEANING BRUSH . . . EVERY 3 MONTHS

NECK ASSEMBLY . . . EVERY 8 MONTHS


Replace integral parts of the Giraffe Bottle Tower System regularly to maintain optimal bottle hygiene and function.



Recommended Use

Which Beverages Can You Drink?

Giraffe Bottle is recommended for water and other **non-viscous** liquids. Remove the check valve when enjoying other beverages.

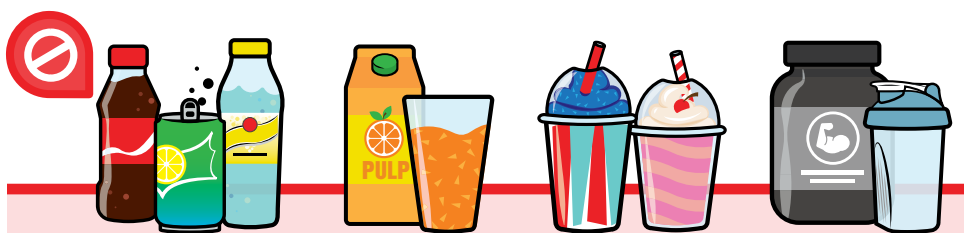


! The check valve may become permanently damaged if used with carbonated drinks

Avoid Leaks

When the straw tip is lower than the liquid level of the bottle, it creates a siphon effect pulling liquids up/out.

Keep your straw tip positioned upward, position the bottle **vertically** as allowed, and replace integral parts of your system periodically to avoid spills and dripping.

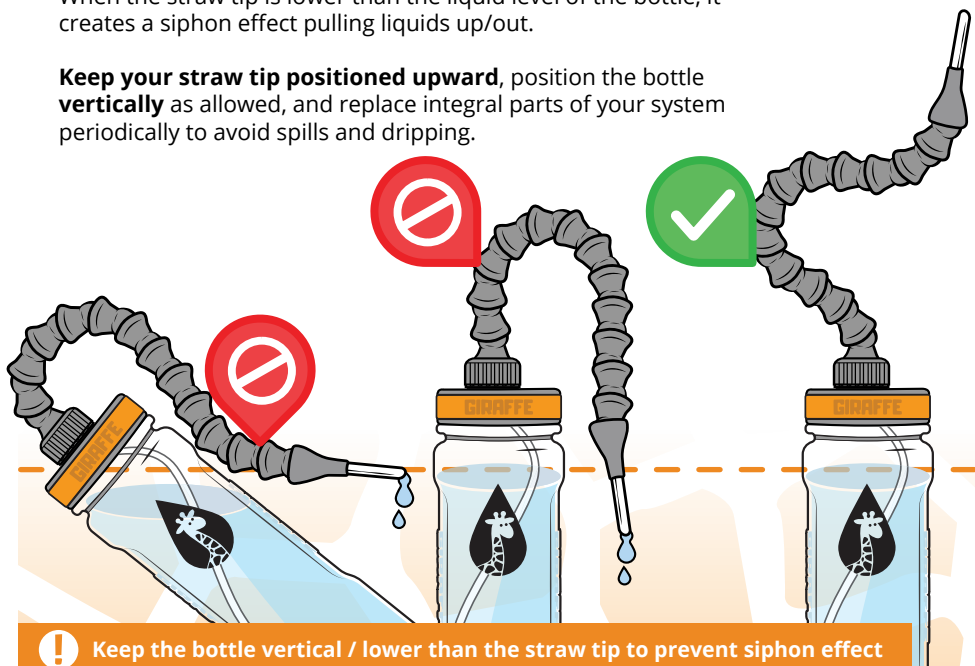

! Keep the bottle vertical / lower than the straw tip to prevent siphon effect

THE TOWER SYSTEM



NECK LENGTHS AVAILABLE

15", 25", 30", 36"
(38 cm, 64 cm, 76 cm, 91 cm)

DRINKING TUBE TYPE

Small, Housed in Modular Plastic
5/32" (4mm) Inner Diameter

FEATURED LIQUID CONTROL


1-Way Check Valve
*Removable from Tube

BOTTLE HOLDER OPTIONS

Aluminum Bottle Holder


THIS SYSTEM IS BEST FOR:

Individuals with little to no upper
body mobility, and mild-to-
moderate reduced lung function.

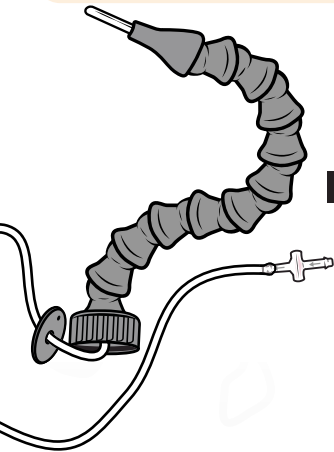


LEVELS OF

HYDRATION

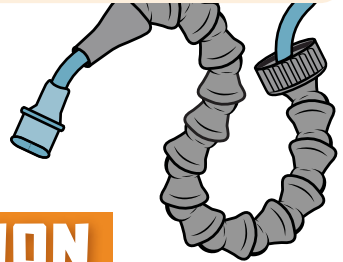

Giraffe Bottle offers a few unique options for Hands-Free Hydration. Each one is tailored for different needs and purposes, with experiences of individuals in mind.

Our systems are cross compatible with every bottle; explore them all and see which fits your needs best!

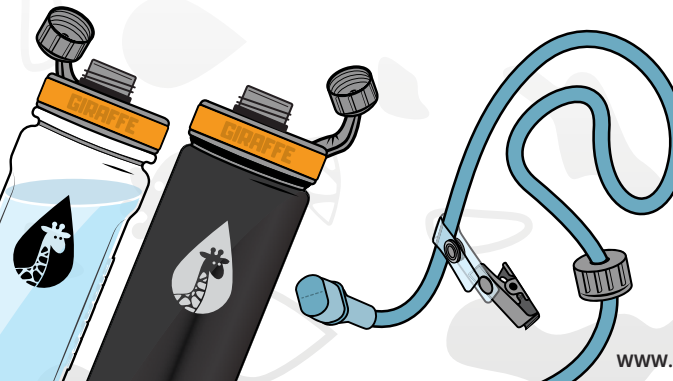

FAQ'S

Questions about your
current system?

WWW.GIRAFFEBOTTLE.COM/PRODUCT-FAQ

LEARNING CENTER

Check out the Learning Center on **GiraffeBottle.com** to explore Hands-Free Hydration, and Frequently Asked Questions about our products. Feel free to contact Giraffe Bottle with any questions, comments, or feedback.



Giraffe Bottle Accessories



Replacement Giraffe Bottles

Tritan Clear: #504111
Smoke Black: #504211
Burnt Orange: #504212
Ltd. Ed. SGN Pattern: #504221



Drinking Tube + Check Valve

For 15" Neck: #504082
For 25" Neck: #504084
For 30" Neck: #504085
For 36" Neck: #504086



Tower Neck Assembly

15" Neck: #504012
25" Neck: #504014
30" Neck: #504015
36" Neck: #504016



Clamp Mount Bottle Holder

Mounts to wheelchairs and other round tubing.

#500311



Cushion Companion Bottle Holder

Slide under mattresses, seat cushions, and other surfaces.

#500332



Weighted Desktop Base

Keep bottles secure and upright on flat surfaces with this sturdy bottle holder.

#500323



Rail Mount Bottle Holder

Attaches to powerchairs with accessory rails (includes Quantum™, Permobil™).

#500314



SHOWN: TOWER TUBE BRUSH, SILICONE BOTTLE BRUSH, SUCTION BULB

More Available Online

For alternate drinking tubes (due to sensitivity in taste), accessories, and other replacement parts, explore online or contact Giraffe Bottle.

WWW.GIRAFFEBOTTLE.COM



GIRAFFE
BOTTLE

a division of:
Oak Hill
BRANDS CORP.

SALES@GIRAFFEBOTTLE.COM

1-630-922-5004

Product designed and assembled in the USA. Manufactured with U.S. and global components.

Information and specifications listed are subject to change at any time without notice.

Permobil, Quantum, and all other brands that may have been mentioned, either directly or indirectly, are trademarks of someone else and are not the property of Oak Hill Brands. Giraffe Bottle is a registered trademark of Oak Hill Brands Corp.

v2305